HEATED BACK BELT SUPPORT

Unisex

CONGRATULATIONS on your purchase of the ObusForme® Back Belt. When used together with a complete back safety training program, your back belt is designed to support your lower back and abdominal muscles to help promote proper posture and good body mechanics while lifting, pushing, pulling, walking, and repetitive bending or twisting.

WARNING


This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety.

Children should be supervised to ensure that they do not play with the appliance.


1. Detach the dual-elastic abdominal panels. Grasp each end of the mesh wrap. Stretch and secure the right end across the left in a comfortable position on the hook and loop closure.

2. Grasp each dual-elastic abdominal panel. Stretch and secure each panel in a comfortable position on the hook and loop closure.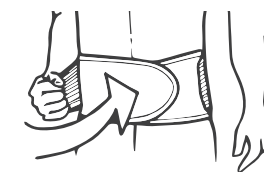

3. The top edge of the back belt should be at or just below your navel. The back belt should not be worn around the midsection of the stomach but should fit snug and low over your hips.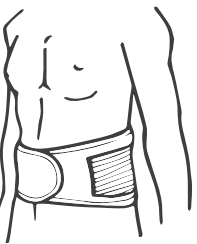

Please Note: Do not rely on your back belt to become a substitute for good lifting, bending, pulling, pushing, or twisting techniques.

Your back belt will not enable you to lift heavier amounts of weight.

Do not subject yourself to any risk by lifting more weight than you can handle.

Only wear your back belt if you are performing tasks where proper posture and good body mechanics are an issue. Continuous wear may cause loss of muscle tone.


CARE FOR YOUR BACK BELT

To Clean:

- Do not machine wash.
- Spot clean with a slightly damp cloth only.
- Do not bleach.
- Lay flat to dry. Do not machine dry.

Instructions for Heating function:

1. Connect the control cable to the power bank and back belt
2. Press the button on the control cable to turn unit on. Press the button again to select temperature.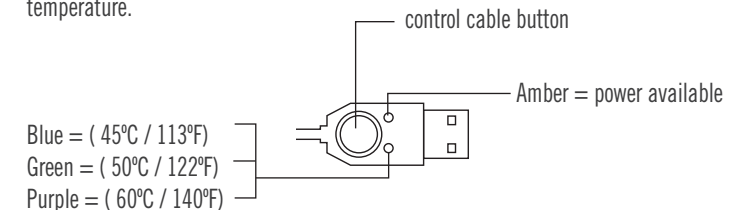

3. When not using the heat setting the back belt can be used for support.

Protective setting: the heating function of the back belt will automatically shutdown after 20 minutes. It must only be supplied at safety extra low voltage corresponding to the marking on the appliance.

Instructions for Charging Power Bank:


1. Use Micro USB (≡) to connect to battery
2. Use USB (≡) to plug into available USB charging port

Powerbank should last up to 4 hours upon full charge. Full charge of Powerbank can take up to 3 hours.

To check remaining power capacity of Power Bank, press the indicator button when the device is not charging. Battery indicator will show the following status:

Power Level Indicator.

- 1 : 0~25%
- 2 : 25~50%
- 3: 50~75%
- 4 : 75~100%

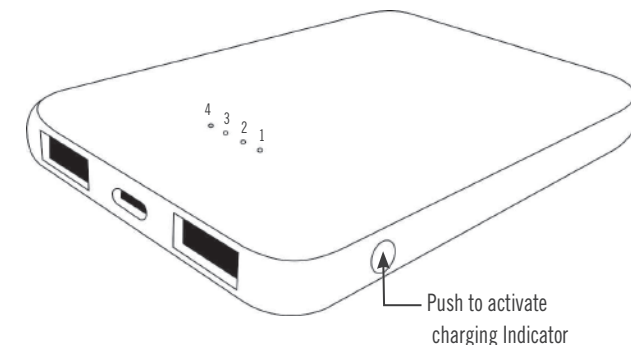Warning: Any discomfort when using the product stop using immediately and consult a physician.

OBUSFORME LIMITED WARRANTY

ObusForme™ guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase . This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme™ by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change.

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme™, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please inform Customer Service by:

Mail: HoMedics Group Canada
6460 Kennedy Road. Unit C
Mississauga, Ontario, Canada L5T 2X4
Telephone: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
www.ObusForme.com

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