# BACK BELTS

Male

Female

Unisex

**Instruction Manual and  
Warranty Information**

BB-ML1-SM/BB-ML1-ML/BB-ML1-LX/BB-ML1-XX  
BB-FM1-SM/BB-FM1-ML/BB-FM1-LX/BB-FM1-XX  
BB-UN1-SM/BB-UN1-ML/BB-UN1-LX

# CONGRATULATIONS


on your purchase of the ObusForme® Back Belt. When used together with a complete back safety training program, your back belt is designed to support your lower back and abdominal muscles to help promote proper posture and good body mechanics while lifting, pushing, pulling, walking, and repetitive bending or twisting.

## APPLYING YOUR BACK BELT


**Helpful Hint:** When applying your belt, try lying down to make sure your stomach is as flat as possible.

1. Wear your back belt either on top of or under clothing.
2. If using a back belt with suspenders, slip them over your shoulders. The suspenders allow your back belt to be worn unfastened when it's not in use. If you do not require the suspenders, you can easily remove them.

3. Detach the dual-elastic abdominal panels. Grasp each end of the mesh wrap. Stretch and secure the right end across the left in a comfortable position on the hook and loop closure. It should not be necessary to over stretch the mesh wrap beyond the hook and loop closure.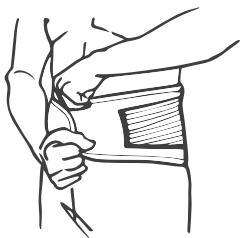


4. Grasp each dual-elastic abdominal panel. Stretch and secure each panel in a comfortable position on the hook and loop closure.

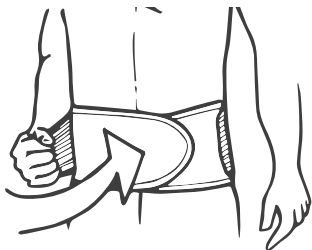5. If using a back belt with suspenders, adjust the suspenders with the buckles. Ensure a comfortable fit without tension.

6. The top edge of the back belt should be at or just below your navel. The back belt should not be worn around the midsection of the stomach but should fit snug and low over your hips.

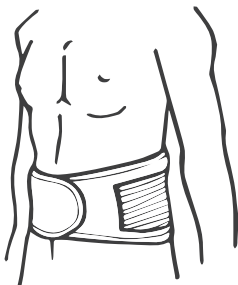**Please Note:** Do not rely on your back belt to become a substitute for good lifting, bending, pulling, pushing, or twisting techniques.

Your back belt will not enable you to lift heavier amounts of weight.

Do not subject yourself to any risk by lifting more weight than you can handle.

Only wear your back belt if you are performing tasks where proper posture and good body mechanics are an issue. Continuous wear may cause loss of muscle tone.

## CARE FOR YOUR BACK BELT

### **To Clean:**

- Spot clean with a damp cloth only.
- Do not bleach.
- Lay flat to dry. Do not machine dry.

## OBUSFORME LIMITED WARRANTY

ObusForme guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change.

## WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

## HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please inform Customer Service by:

Mail: HoMedics Group Canada  
6460 Kennedy Road. Unit C

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