OBUSFORME®

Professional

ENGINEERED FOR THE HUMAN BODY™

CUSTOMAIR® BACKREST SUPPORT WITH ADJUSTABLE LUMBAR

CL-PROF-01

CONGRATULATIONS

Thank you for your purchase of the ObusForme® CustomAIR Backrest Support with Adjustable Lumbar. The award-winning ObusForme® Backrest Supports provide support and comfort for your back to assist in the prevention and relief of back pain. The only backrest exclusively endorsed by the Canadian Chiropractic Association, it contains an "S"-shaped frame that conforms to the contours of the back to guide the spine into correct alignment and place you in a proper sitting posture. The ObusForme® CustomAIR® Backrest features an internal, inflatable lumbar support system. Lightweight and portable, each backrest support turns any chair into an ergonomic seating system; they can be used in the home, office, vehicle, or anywhere else you sit.

WARNING

Before using your ObusForme CustomAIR Backrest Support in a vehicle, please consult the owner's manual to ensure that you can properly adjust your vehicle's seat and/or headrest after installation of the ObusForme CustomAIR Backrest Support.

USING THE ADJUSTABLE LUMBAR SUPPORT SYSTEM

IMPORTANT NOTE: ALWAYS RELEASE THE AIR BEFORE PERFORMING ANY ADJUSTMENTS ON THE VERTICAL LOCATION OF THE LUMBAR SUPPORT BLADDER. See DEFLATE section below for instructions on deflating.

The internal inflatable lumbar support system provide added support for the lower spine by filling in the lower inward curve of your back. For custom support to find the lumbar height that fits your comfort position, pull the top strap to raise the lumbar support. Pull the strap down to lower the lumbar support. Once you have located the best position vertically, pump up the support using the bulb that is conveniently stored in the velcro pouch on the side of the backrest. Lumbar supports can be used for everyday use as extra lower back support or to reduce spinal stress in people with a marked lumbar lordosis or sway-back.

Please Note: It is not necessary to use the internal inflatable lumbar support system on your ObusForme® CustomAIR® Backrest Support. If the internal inflatable lumbar support system feels bulky or uncomfortable, you may not have an excessive lumbar curve that requires extra support. Adjust the lumbar support system up and down as well as inflation or deflation level to find the suitable amount of comfort for you.

The inflation pump bulb and tube should be stored in the pouch when the desired lumbar support has been achieved.

INFLATING AND DEFLATING THE LUMBAR SUPPORT

INFLATE: Rest your lower back against the CustomAIR®. From full deflation, there are approximately 50 comfort positions for the internal inflatable lumbar support system. When you have reached the most comfortable position for your back, simply stop pumping. The level of air in the lumbar support will remain fixed indefinitely until changed.


DEFLATE: Press the deflate button, that is located above the bulb at the end of the inflation tube.

CLEANING THE CUSTOMAIR BACKREST

The ObusForme® CustomAIR® Backrest cover is made of synthetic materials. It's easy to clean with a mild soap and a sponge, damp cloth, or soft brush. Do not rub the cover excessively or place it in a washing machine.

Please Note: The ObusForme® CustomAIR® Backrest may in some cases require time to get used to. Over the years your spine has become less flexible, and needs time to adjust to its natural shape again. If you feel discomfort when using your ObusForme® CustomAIR® Backrest for the first time, it is recommended that you use it for shorter periods of time, gradually increasing your usage until you feel comfortable.

If you have a very sore back or small curvature (lumbar area), it is highly recommended that you allow yourself a break-in period of at least 30 days in order to obtain the full benefits of your ObusForme® CustomAIR® Backrest.

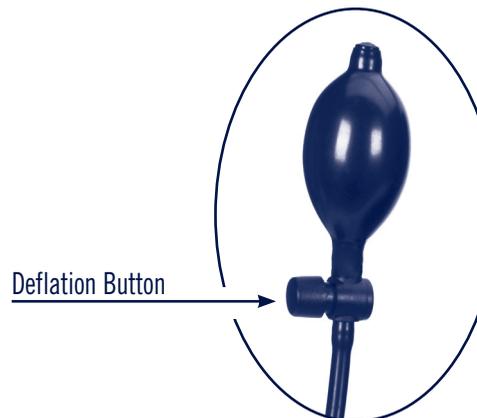OBUSFORME® LIMITED WARRANTY

(CANADA Only)

ObusForme guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt with the defective product. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change.

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please contact Customer Service by:

HoMedics Group Canada
6460 Kennedy Road, Unit C
Mississauga, Ontario, Canada L5T 2X4
Tel: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
CustomerServiceCAN@obusforme.com
www.obusforme.com

JUL. 2014

Register your product (s) to ensure faster and more efficient customer service. It will allow us to keep you up-to-date with warranty and product information. Please take a few minutes to register online now.

Register at
<http://www.obusforme.com/product-registration/>

OBUSFORME®

Corporate Member of ACE
ACE

ASSOCIATION OF CANADIAN ERGONOMISTS

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Canadian
Chiropractic
Association

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