OBUSFORME[®]
ENGINEERED FOR THE HUMAN BODY

HOT/COLD KNEE SUPPORT

INSTRUCTION MANUAL AND WARRANTY INFORMATION

ONE YEAR LIMITED WARRANTY

GS-HC-KN

HOT/COLD WRIST SUPPORT

WARNING

Heated gel insert may cause serious burns if too hot. Do not use over sensitive skin or where there is poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

CAUTION

- Do not insert the gel pack directly into microwave as this may cause damage to the seal.
- Do not use the heated or frozen gel pack for more than 20 minutes at a time.
- Consult a physician prior to use.
- If the contents of the gel pack come in contact with eyes, flush eyes with water. Contact a physician if irritation persists.
- Never apply a heated or frozen gel pack directly to skin; it must be either be wrapped in a towel or placed in the designated pocket of the Back Belt. If the heated or frozen gel pack is uncomfortable during use, remove it immediately.
- Do not use the gel pack with sport creams, balms, liniments or lotions.
- Children should not use or play with the gel pack
- Use the heated pack carefully as it may cause severe burns.
- This product, while non-toxic, is not intended for ingestion. If contents are swallowed, do not induce vomiting. Consult a physician.
- Do not use heated or frozen packs over skin areas with less than normal sensitivity.
- Do not use heated gel insert on parts of body where there is swelling, redness or immediately after an injury. Cold is recommended for these conditions.
- Do not use the gel pack over open wounds.

INSTRUCTIONS FOR USE

Gel Pack – Cold Application

For use as a cold pack, place the reusable gel pack in a freezer for a minimum of 4 hours to achieve optimal temperature and performance. The gel pack will remain flexible even when frozen. To use, remove from the freezer and insert in the pocket at the back of the back belt.

Gel Pack – Hot Application

For use as a heat pack, the reusable gel pack is heated by placing it in a container of hot water. The water may be heated in a kettle, on a stovetop or in a microwave oven.

Kettle or Stovetop

1. In the case of a kettle, pour enough heated water into a suitable container to easily cover the gel pack. For stovetop heating, pour an appropriate amount of water into a large pan. Bring water to a boil and then remove from heat.
2. Place the gel pack in hot water for 1 minute. Do not boil the gel pack directly.
3. Remove the gel pack using tongs. Use caution as the pack will be very hot.
4. Test the temperature of the gel pack by placing it briefly on the underside of your wrist. If the pack is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until the desired temperature is reached.
5. Place the gel pack in the pocket at the back of the wrist wrap.

Microwave

(Caution: Do not heat the gel insert directly in a microwave oven!)




1. Heat water in microwaveable container on high setting for 3 minutes. This is based on a 1000 watt microwave. Use a minimum of 2 cups or 0.5 litres of water. Heating times may vary. Follow all precautions and recommendations found in your microwave oven instruction manual with respect to the heating of water. Do not insert the gel pack directly into microwave as this may cause damage to the seal. After heating, let water stand in microwave for thirty seconds before moving it or inserting the gel pack.
2. Place the gel pack in hot water for 1 minute.
3. Remove the pack using tongs. Use caution as the gel pack will be very hot.
4. Test the temperature of the gel pack by placing it briefly on the underside of your wrist. If the pack is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until the desired temperature is reached.
5. Place the gel pack in the pocket at the back of the back wrap.

CARE INSTRUCTION

Remove the gel insert before washing the back belt. Spot clean with a damp cloth only; hand to dry.

USING YOUR HOT/COLD KNEE SUPPORT

1. Heat/freeze gel pack for desired use
2. Place gel pack into knee support
3. Use fastener straps to secure support in place

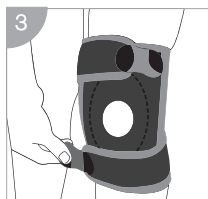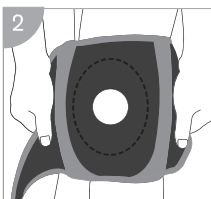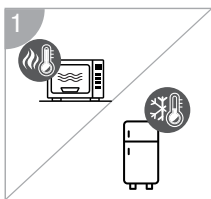OBUSFORME LIMITED WARRANTY

ObusForme[®] guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme[®] by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change.

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme[®], shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please contact Customer Service by:

HoMedics Group Canada
6460 Kennedy Road. Unit C
Mississauga, Ontario, Canada L5T 2X4
Tel: (416) 785-1386 • Fax: (416) 785-5862 • Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
cservice@obusforme.com
www.obusforme.com

Register your product (s) to ensure faster and more efficient customer service. It will allow us to keep you up-to-date with warranty and product information. Please take a few minutes to register online. Register at: <http://www.obusforme.com>

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