OBUSFORME®

CustomAIR® SEAT CUSHION

English, Français

CONGRATULATIONS

on your purchase of the ObusForme CustomAIR Seat Cushion. The CustomAIR Seat Cushion's easy-to-use self-inflating technology allows for outstanding portability compared to fixed seating designs.

About your ObusForme® CustomAIR Seat Cushion

The unique design of the cushion's open cell foam and sealed cushion envelope allows for adjustable pressure relief and weight distribution for personalization according to the user's preferences. Use alone, or with an ObusForme backrest for full comfort and support.

Lightweight and portable, the CustomAIR Seat Cushion is ideal for travel, sporting events, camping or anywhere truly portable seating comfort is desired.

How to use your ObusForme® CustomAIR Seat Cushion

1. Remove the seat cushion from its carry pouch and unroll it.
2. To inflate the cushion: The ObusForme CustomAIR Seat Cushion is designed to inflate automatically when its air valve is in the open position. To inflate, twist the air valve clockwise, allowing air to flow into the cushion through the valve. When inflation is complete, lock the air valve into its closed position by turning it counter-clockwise.

Note that depending upon how long the cushion has been stored in its rolled state and the ambient air temperature, inflation of the cushion may take a minute or so to complete and may require you to physically force more air into the cushion if additional firmness or support is desired. For instructions on adding air to the cushion, see point #3 below.

3. To adjust the level of support:
 - To increase the level of firmness or support, unlock the air valve by turning it clockwise, press the air valve inwards until the desired amount of air has flowed into the cushion, then close the air valve. To force more air into the cushion (if desired), simply place your mouth over the air valve and blow into it as you would a balloon until the cushion has reached the desired level of firmness, then quickly turn the air valve to its locked position.
 - To decrease the level of firmness or support, unlock the air valve by turning it clockwise, press the air valve inwards and gently squeeze the cushion until the desired amount of air has flowed out of the cushion, then close the air valve.
4. To deflate the cushion: Unlock the air valve by turning it clockwise and roll the cushion up tightly, starting at the end opposite the air valve. When the cushion has been fully rolled up, close the air valve and the cushion will stay in its rolled form. Replace the cushion in its carry pouch.

NOTE: Immediately after purchase or a period of disuse, the cover of the cushion may have a slightly wrinkled appearance. This is a function of it having been stored tightly rolled in its carry pouch for an extended period of time. This wrinkled appearance will diminish in time if the cushion is left inflated and will not affect its performance or level of comfort.

OBUS FORME LIMITED WARRANTY

(North America Only)

ObusForme® guarantees all items are free from defects in workmanship & materials for a time stated below from the original purchase date. This applies when items are used for the purpose intended. Items will be repaired/replaced with new/refurbished parts/items &/or alternates (our option) if the ORIGINAL purchaser has sent the completed Warranty Registration within 30 days of purchase & includes original receipt & item. Shipping, customs, duties and taxes must be PRE-PAID TO and FROM ObusForme® by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change. This warranty is non-transferable.

WHAT IS NOT COVERED

Wear & tear, aging, foam/item discoloring, odor, flattening, density, variation, leaking, alteration, mishandling, faulty adjustment, misuse, improper care, power damage, accidents, rental use, obsolete items, service by anyone other than ObusForme®, use of any non-ObusForme® authorized parts, shipping damage, over-inflation, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

WARRANTY TIME FRAME

Backrest Frame: Lifetime
(cover, foam, lumbar pad and other Backrest parts/materials are NOT covered)

Seat Frame: 1 year
(cover, foam and other Seat parts/materials are NOT covered)

Back Therapy: 1 year
(includes but is NOT limited to Back Belts, Backpacks, Drivers Seats)

Sleep/Foot/Muscle Therapy: 1 year
(pillow case is NOT included & must not be returned. Only foam or bladder will be replaced.)

Electrical Parts: 1 year
(This includes wires, adaptors, plugs and other electrical parts/components)

HOW TO OBTAIN WARRANTY SERVICE

You must obtain Return Authorization & direction before sending your item or it will be DENIED. Please obtain an RA via Customer Service by:

HoMedics Group Canada
344 Consumers Road
Toronto, Ontario M2J 1P8
Tel/Tél : (416) 785-1386
Fax/ Téléc : (416) 785-5862
Toll Free/ Sans Frais : 1-888-225-7378
www.obusforme.com

OBUSFORME®

ASSOCIATION OF CANADIAN ERGONOMISTS

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ObusForme® Ltd. is committed to providing you with optimal relief and comfort. To serve you better in the future, we would like to know if we have fulfilled our commitment. Please complete and return this Questionnaire to help us better meet your needs.

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We aggregate this information and use it internally for research and marketing purposes only. We do not disclose personal information to any third parties. If you have any questions about the personal information that we keep on file, please contact a customer service representative at the number listed below.

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5. Any additional comments / Autres commentaires :

Please sign below if you consent to our use, in any media, of your name together with all or part of your comments about ObusForme® Ltd. or our products in our advertising and promotional material.

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