OBUSFORME®
ENGINEERED FOR THE HUMAN BODY™

ULTRAFORME™ UNIVERSAL BACKREST

English, Français

UF-BLK-UNI

CONGRATULATIONS

on your purchase of the UltraForme™ Universal Backrest. Our unique convex and concave design helps align your spine into its proper curvature, as well as aiding in the relief and prevention of back problems. The adjustable UltraForme Universal Backrest offers custom back support to people of all sizes.

SCIENTIFICALLY TESTED

The UltraForme Universal Backrest relieves back pain by offering complete spinal support while minimizing damaging pressure points and disc pressure. Properly used, it will also significantly reduce muscle strain in the lower back and increase respiratory capacity.

For optimal results, use the UltraForme Universal Backrest with the ObusForme Contoured Seat Cushion to create a complete, comfortable and ergonomic sitting experience for your home, office or car. Together, these unique products encourage proper alignment between the pelvis and spine and further reduce pressure in the back, providing even more comfort for you each and every day.

USING YOUR OBUSULTRAFORME BACKREST

1. Unfasten the buckles at the back of the backrest and place the backrest against the back of the seat/ chair where the device will be used.
2. Sit back and align your back against the UltraForme Universal Backrest. Move the backrest up and down until the curvature of your UltraForme Universal Backrest most comfortably meets your lumbar curvature (the small of your back).
3. Hold the backrest in place and turn around to make note of the distance between the spot where the base of the backrest meets the back of the chair and

the seat of the chair. If necessary, adjust the length of the slider between the top and bottom portions of the backrest so that the base of the backrest sits on the seat cushion and the backrest most comfortably meets your lumbar curvature.

Should you need to position the backrest higher than the adjustable base permits, adjust the slider position to whatever length suits you best and use the two quick-connect straps at the rear of the backrest to tightly secure the backrest to the chair in your ideal position.

4. Adjust the position of the back of your chair by tilting it backwards or forward as necessary.
5. Before using your UltraForme Universal Backrest in a vehicle, please consult the owner's manual to ensure that you can properly adjust your vehicle's seat and/or headrest after installation of the UltraForme Universal Backrest.

Please Note: The UltraForme Universal Backrest may in some cases require time to get used to. Over the years, your spine has become less flexible and needs time to adjust to its natural shape again. If you feel discomfort when using your UltraForme Universal Backrest for the first time, it is recommended that you use it for shorter periods of time, gradually increasing your usage until you feel comfortable. If you have a very sore back or small curvature (lumbar area), it is highly recommended that you allow yourself a break-in period of at least 30 days in order to obtain the full benefits of your UltraForme Universal Backrest. Those with scoliosis or a history of disc fusion surgery may not get the full benefit of the UltraForme Universal Backrest. For these individuals, we recommend professional advice to determine if the ObusForme® Lowback Backrest Support is right for you.

EASY CARE

Most cleaning can be easily accomplished with the use of a mild soap and water solution. However, we do suggest that you do a spot cleaning trial on an inconspicuous area. Do not rub the cover or immerse the entire cover in water as it may shrink.

ADJUSTING YOUR BACKREST LENGTH

CLOSED
POSITION
(Shortest Height)

EXTENDED
POSITION
(Tallest Height)

OBUSFORME® LIMITED WARRANTY

(CANADA Only)

ObusForme guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, please return this product to ObusForme for refund or replacement. This warranty gives you rights that vary by province/state. This warranty may change.

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to returning your item or it may be denied. Please contact Customer Service by:

Telephone: 1-888-225-7378 or 416-785-1386
Mon – Fri, 8:30 a.m. to 5:00 p.m., EST

Fax: (416) 785-5862

Email: CustomerServiceCAN@obusforme.com

Mail:
HoMedics Group Canada
6460 Kennedy Road, Unit C
Mississauga, Ontario, Canada L5T 2X4

DEC. 2017

Register your product (s) to ensure faster and more efficient customer service. It will allow us to keep you up-to-date with warranty and product information. Please take a few minutes to register online now.

Register at
<http://www.obusforme.com/product-registration/>

OBUSFORME®

Corporate Member
Membre Corporatif
ACE

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Canadian
Chiropractic
Association

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<http://www.obusforme.com/product-registration/>

OBUSFORME®

Corporate Member
Membre Corporatif

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