DOSSIER
OBUS *ultra* **FORME**TM
BACKREST

The New Science in Back Pain ReliefTM
**La science au service du soulagement
des maux de dos^{MC}**

Congratulations on your purchase of the Obus Ultra FormeTM Backrest. Our unique convex and concave design helps align your spine into its proper curvature, as well as aiding in the relief and prevention of back problems. With three different sizes to choose from, the Obus Ultra Forme Backrest offers back pain relief to people of all sizes.

Scientifically tested, the Obus Ultra Forme Backrest relieves back pain by offering complete spinal support while minimizing damaging pressure points and disc pressure. Properly used, it will also significantly reduce muscle strain in the lower back and increase respiratory capacity.

For optimal results, use the Obus Ultra Forme Backrest with the Obus Ultra SeatTM to create a complete, comfortable and ergonomic sitting experience for your home, office or car. Together, these unique products encourage proper alignment between the pelvis and spine and further reduce pressure in the back, providing even more comfort for you each and every day.

Using your Obus Ultra Forme Backrest

1. Place the Obus Ultra Forme Backrest as far back as possible against the back of the seat/chair where the device will be used.
2. Sit back and align your back against the Obus Ultra Forme Backrest. Adjust the device so the curvature of your Obus Ultra Forme Backrest most comfortably meets your lumbar curvature (the small of your back).
3. The Obus Ultra Forme Backrest stays in position when the elastic straps are placed or stretched around most chairs or vehicle seats. Tighten the straps using the adjustable buckles.
4. Adjust the position of the back of your chair by tilting it backwards or forward as necessary.
5. Before using your Obus Ultra Forme Backrest in a vehicle, please consult the owner's manual to ensure that you can properly adjust your vehicle's seat and/or headrest after installation of the Obus Ultra Forme Backrest.

Please Note: The Obus Ultra Forme Backrest may in some cases require time to get used to. Over the years, your spine has become less flexible and needs time to adjust to its natural shape again. If you feel discomfort when using your Obus Ultra Forme Backrest


for the first time, it is recommended that you use it for shorter periods of time, gradually increasing your usage until you feel comfortable. If you have a very sore back or small curvature (lumbar area), it is highly recommended that you allow yourself a break-in period of at least 30 days in order to obtain the full benefits of your Obus Ultra Forme Backrest. Those with scoliosis or a history of disc fusion surgery may not get the full benefit of the Obus Ultra Forme Backrest. For these individuals, we recommend professional advice to determine if the ObusForme[®] Lowback Backrest Support is right for them.

Easy Care

Most cleaning can be easily accomplished with the use of a mild soap and water solution. However, we do suggest that you do a spot cleaning trial on an inconspicuous area. Do not rub the cover or immerse the entire cover in water as it may shrink.

Unconditional 30-Day Money-Back Guarantee

If at any time during the first 30 days after purchase the Obus Ultra Forme Backrest fails to meet your expectations, your money will be refunded. Simply RETURN TO THE PLACE OF PURCHASE WITH ORIGINAL RECEIPT.

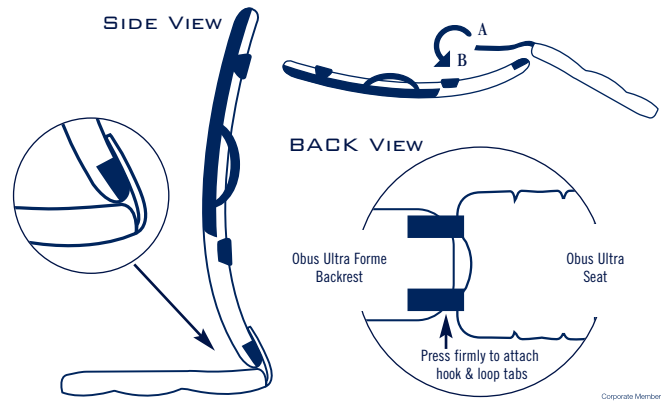ASSOCIATION OF CANADIAN ERGONOMISTS

LIMITED WARRANTY: ObusForme[®] guarantees all items are free from defects in workmanship and materials for a time stated below from the original purchase date. This applies when items are used for the purpose intended. Items will be repaired/replaced with new/refurbished parts/items and/or alternates (our option) if the ORIGINAL purchaser has sent the completed Warranty Registration within 30 days of purchase and includes ORIGINAL RECEIPT and item. Shipping, customs, duties and taxes must be PRE-PAID TO and FROM ObusForme[®] by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change. This warranty is non-transferable.

WHAT IS NOT COVERED: Wear and tear, aging, foam/item discoloring, odor, flattening, density, variation, leaking, alteration, mishandling, faulty adjustment, misuse, improper care, power damage, accidents, rental use, obsolete items, service by anyone other than ObusForme[®], use of any non-ObusForme[®] authorized parts, shipping damage, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

WARRANTY TIME FRAME: Backrest Frame: Lifetime (cover, foam, lumbar pad and other Backrest parts/materials are NOT covered)
Seat Frame: 1 year (cover, foam and other Seat parts/materials are NOT covered)

HOW TO OBTAIN WARRANTY SERVICE

You must obtain a Return Authorization (RA) and reduction before sending your item or it will DENIED. Please obtain an RA via Customer Service by:

Mail: HoMedics Group Canada
344 Consumers Road, Toronto, Ontario, Canada, M2J1P8
Tel: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri ET
www.obusforme.com

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ObusForme[®] is committed to providing you with optimal relief and comfort. To serve you better in the future, we would like to know if we have fulfilled our commitment. Please complete and return this Questionnaire to help us better meet your needs.

We aggregate this information and use it internally for research and marketing purposes only. We do not disclose personal information to any third parties. If you have any questions about the personal information that we keep on file, please contact a customer service representative at the number listed below.

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