OBUSFORME®
ENGINEERED FOR THE HUMAN BODY™

BACK BELT

WITH HOT & COLD
GEL PACK

BB-HCBB-ML

**Instruction Manual and
Warranty Information**

Thank you for purchasing of the OBUSFORME® Back Belt with Hot & Cold Gel Pack. When used together with a complete back safety training program, your back belt is designed to support your lower back and abdominal muscles while lifting, pushing, pulling, walking, and repetitive bending or twisting. The included Hot/Cold Gel Pack can provide the additional benefits of targeted heat or cold while you wear your back support belt.

WARNING

Heated gel insert may cause serious burns if too hot. Do not use over sensitive skin or where there is poor circulation. The unattended use of heat by children or incapacitated persons may be dangerous.

CAUTION

- Do not insert the gel pack directly into microwave as this may cause damage to the seal.
- Do not use the heated or frozen gel pack for more than 20 minutes at a time.
- Consult a physician prior to use.
- If the contents of the gel pack come in contact with eyes, flush eyes with water. Contact a physician if irritation persists.
- Never apply a heated or frozen gel pack directly to skin; it must be either be wrapped in a towel or placed in the designated pocket of the Back Belt. If the heated or frozen gel pack is uncomfortable during use, remove it immediately.
- Do not use the gel pack with sport creams, balms, liniments or lotions.
- Children should not use or play with the gel pack
- Use the heated pack carefully as it may cause severe burns.
- This product, while non-toxic, is not intended for ingestion. If contents are swallowed, do not induce vomiting. Consult a physician.
- Do not use heated or frozen packs over skin areas with less than normal sensitivity.
- Do not use heated gel insert on parts of body where there is swelling, redness or immediately after an injury. Cold is recommended for these conditions.
- Do not use the gel pack over open wounds.

INSTRUCTIONS FOR USE

Back Belt Fitting Instructions

The ObusForme Back Belt actually has two “belts”, inner and outer, to provide a secure fit. The inner belt is larger, while the outer belt is smaller and more elastic. The back belt is best fitted while standing. It is most effective when used directly against the skin, but may also be worn over clothing. The inside of the back belt should face your body.

1. Detach the two ends of the outer belt from the inner belt. Wrap the inner belt around your back and secure in front with the fastening material. While fitting, ensure the outer belt is not fastened to the outside of inner belt, as this will affect the fit. When fastened, the inner belt should fit comfortably and securely without restricting breathing or circulation.
2. Bring one side of outer belt around and stretch firmly in front of your body. Attach fastener. Repeat with the remaining side.

Gel Pack – Cold Application

For use as a cold pack, place the reusable gel pack in a freezer for a minimum of 4 hours to achieve optimal temperature and performance. The gel pack will remain flexible even when frozen. To use, remove from the freezer and insert in the pocket at the back of the back belt.

Gel Pack – Hot Application

For use as a heat pack, the reusable gel pack is heated by placing it in a container of hot water. The water may be heated in a kettle, on a stovetop or in a microwave oven.

Kettle or Stovetop

1. In the case of a kettle, pour enough heated water into a suitable container to easily cover the gel pack. For stovetop heating, pour an appropriate amount of water into a large pan. Bring water to a boil and then remove from heat.
2. Place the gel pack in hot water for 1 minute. Do not boil the gel pack directly.

3. Remove the gel pack using tongs. Use caution as the pack will be very hot.
4. Test the temperature of the gel pack by placing it briefly on the underside of your wrist. If the pack is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until the desired temperature is reached.
5. Place the gel pack in the pocket at the back of the back wrap.

Microwave

(Caution: Do not heat the gel insert directly in a microwave oven!)

1. Heat water in microwaveable container on high setting for 3 minutes. This is based on a 1000 watt microwave. Use a minimum of 2 cups or 0.5 litres of water. Heating times may vary. Follow all precautions and recommendations found in your microwave oven instruction manual with respect to the heating of water. Do not insert the gel pack directly into microwave as this may cause damage to the seal. After heating, let water stand in microwave for thirty seconds before moving it or inserting the gel pack.
2. Place the gel pack in hot water for 1 minute.
3. Remove the pack using tongs. Use caution as the gel pack will be very hot.
4. Test the temperature of the gel pack by placing it briefly on the underside of your wrist. If the pack is too hot, allow it to cool before using. If it is not hot enough, continue heating in 1 minute intervals until the desired temperature is reached.
5. Place the gel pack in the pocket at the back of the back wrap.

CARE INSTRUCTION

Remove the gel insert before washing the back belt. Spot clean with a damp cloth only; hand to dry.

DURABLE, QUALITY CONSTRUCTION FOR YEARS OF DEPENDABLE USE

DURABLE ELASTIC ADJUSTMENTS

Allow for customized applications and fit

PREMIUM QUALITY NEOPRENE

Retains natural body heat while
providing excellent support


COMFORTABLE, LIGHTWEIGHT DESIGN

Wear it right under or over
your clothes.

CONVENIENT HOOK & LOOP CLOSURES

For quick and easy fastening

GEL PACK

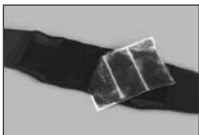Slips easily into
insulated pocket

OBUSFORME LIMITED WARRANTY

ObusForme guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change.

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please inform Customer Service by:

Mail: HoMedics Group Canada
6460 Kennedy Road. Unit C
Mississauga, Ontario, Canada L5T 2X4
Telephone: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
www.ObusForme.com

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