# OBUSFORME®

## The OBUSFORME BACK BELT

# OBUSFORME®

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OBUSFORME

### C O N G R A T U L A T I O N S

on your purchase of the ObusForme® Back Belt. When used together with a complete back safety training program, your back belt is designed to support your lower back and abdominal muscles to help promote proper posture and good body mechanics that help maintain a healthy back, reduce back pain, and prevent back pain injuries due to lifting, pushing, pulling, walking, and repetitive bending or twisting.

#### APPLYING YOUR BACK BELT

Helpful Hint: When applying your belt, try lying down to make sure your stomach is as flat as possible.

1. Please refer to the diagram on the back of your back belt packaging.
2. Wear your back belt either on top of or under clothing.
3. If using a back belt with suspenders, slip them over your shoulders. The suspenders allow your back belt to be worn unfastened when it's not in use. If you do not require the suspenders, you can easily remove them.
4. Detach the dual-elastic abdominal panels. Grasp each end of the mesh wrap. Stretch and secure the right end across the left in a comfortable position on the hook and loop closure. It should not be necessary to over stretch the mesh wrap beyond the hook and loop closure.

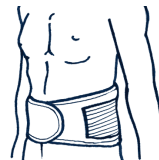
5. Grasp each dual-elastic abdominal panel. Stretch and secure each panel in a comfortable position on the hook and loop closure.

6. If using a back belt with suspenders, adjust the suspenders with the buckles. Ensure a comfortable fit without tension.

7. The top edge of the back belt should be at or just below your navel. The back belt should not be worn around the midsection of the stomach but should fit snug and low over your hips.

**Please Note:** Do not rely on your back belt to become a substitute for good lifting, bending, pulling, pushing, or twisting techniques. Your back belt will not enable you to lift heavier amounts of weight. Do not subject yourself to any risk by lifting more weight than you can handle. Only wear your back belt if you are performing tasks where proper posture and good body mechanics are an issue. Continuous wear may cause loss of muscle tone.

#### CARE FOR YOUR BACK BELT

##### To Clean:

- Hand wash with mild soap and cold water.
- Do not bleach.
- Lay flat to dry. Do not machine dry.

### LIMITED WARRANTY ( IN NORTH AMERICA ONLY )

ObusForme guarantees that all ObusForme® products are free from defects in workmanship and materials throughout the life of the original owner. Any product found to be defective will be replaced or repaired (at our option) without charge. Any product that is no longer in production and/or cannot be repaired will be replaced with a comparable ObusForme® product.

Purchaser must complete and return the Warranty Registration within 30 days of purchase.

To obtain service under this warranty, please contact our Customer Service Department for return authorization. Once authorization is received, please return the product to ObusForme at our address below, together with proof of purchase and prepaid return shipping costs.

#### Exclusions:

Shipping costs to and from ObusForme are the responsibility of the owner. This warranty applies to damages resulting from manufacturer's defects only. Normal wear and tear resulting from natural aging of the product and damages caused by accident, unreasonable use, over inflation of lumbar support, where applicable, negligence or any other cause not resulting from the defects in materials and workmanship, are excluded from this warranty.

This warranty is non-transferable. This warranty gives you specific rights and you may also have other rights which vary from province to province and state to state.

For further information, please contact us at:

HoMedics Group Canada  
344 Consumers Road  
Toronto, Ontario  
Canada M2J 1P8

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ObusForme Ltd. is committed to providing you with optimal relief and comfort. To serve you better in the future, we would like to know if we have fulfilled our commitment. Please complete and return this Questionnaire to help us better meet your needs.

We aggregate this information and use it internally for research and marketing purposes only. We do not disclose personal information to any third parties. If you have any questions about the personal information that we keep on file, please contact a customer service representative at the number listed below.

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