**Thank you for purchasing this ObusForme heat therapy product.**

This product contains all-natural buckwheat seeds which, after heating in the microwave according to the below directions, generate soothing warmth at a steady rate for a period of 20-30 minutes. At room temperature the seeds naturally absorb atmospheric moisture and then release it upon heating, providing the added usage benefit of moist heat comfort. This product can be reused again and again as needed with the same beneficial results.

---

**IMPORTANT SAFETY NOTES:**

Heat therapy should be applied for no more than 20-30 minutes at a time. Comfort level and heat tolerance should be used as guidelines during use. If you feel discomfort or pain, discontinue use immediately. Never use this product on broken or injured skin or areas of the body having a loss of sensitivity.

If you have concerns regarding your health, please consult your physician before use. An injury that is very painful or does not resolve itself after 24-48 hours of self-care should be checked by a medical professional. Individuals with circulatory or diabetic problems should consult a physician before using this product. Use only as directed.

---

**Heating Instructions**

**Note:** This product may *only* be heated in the microwave.

Do not place in boiling water or in any kind of conventional oven.

1. Place the wrap in the microwave and heat it on HIGH for 60 seconds.
2. Remove the wrap from the microwave and knead it gently so as to allow for even heating of the wrap.
3. Replace the wrap in the microwave and heat it for another 30-60 seconds.
4. Remove and apply the wrap to the area of the body requiring heat therapy.

**Please note** that the above timing should be used as a general guideline and applies to microwave ovens of average wattage. ***You should always test the heat level gently with your wrist before placing the wrap on any area of your body.*** If the wrap is uncomfortably warm to hold to your wrist for more than 5-10 seconds, it is likely too hot to apply to any other area of your body.

**Care Instructions**

Spot clean the product's cover as needed using a damp cloth. Do not use detergent and do not machine wash or dry clean the product. Do not immerse the product in water or other liquid.

HoMedics Group Canada  
344 Consumers Road  
Toronto, Ontario, Canada M2J 1P8  
Tel: (416) 785-1386 Fax: (416) 785-5862  
Toll Free: 1-888-225-7378  
8:30 a.m. to 5:00 p.m., Mon - Fri EST  
[www.obusforme.com](http://www.obusforme.com)

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