OBUSFORME®

CUSTOMAIR® BACKREST SUPPORT

CONGRATULATIONS

Thank you for your purchase of the ObusForme® CustomAIR Backrest Support. The award-winning ObusForme® Backrest Supports provide support and comfort for your back to assist in the prevention and relief of back pain.

The only backrest exclusively endorsed by the Canadian Chiropractic Association, it contains an "S"-shaped frame that conforms to the contours of the back to guide the spine into correct alignment and place you in a proper sitting posture. The ObusForme® CustomAIR® Backrest Support features an internal, inflatable lumbar support system. Lightweight and portable, each backrest support turns any chair into an ergonomic seating system; they can be used in the home, office, vehicle, or anywhere else you sit.

Before using your ObusForme® CustomAIR Backrest Support in a vehicle, please consult the owner's manual to ensure that you can properly adjust your vehicle's seat and/or headrest after installation of the ObusForme® CustomAIR Backrest Support.

QUESTIONS & ANSWERS

Q. How does the ObusForme® CustomAIR® Backrest Support fasten to a seat or chair?

A. The ObusForme® CustomAIR® Backrest Support stays in position when the elastic stretch loop is placed or stretched over most chairs or vehicle seats. The stretch loop is not necessary on couches or sofas.

Q. Why should I use the internal inflatable lumbar support system?

A. The internal inflatable lumbar support system provide added support for the lower spine by filling in the lower inward curve of your back. Lumbar supports can be used for everyday use as extra lower back support or to reduce spinal stress in people with a marked lumbar lordosis or sway-back.

Please Note: It is not necessary to use the internal inflatable lumbar support system on your ObusForme® CustomAIR® Backrest Support. If the internal inflatable lumbar support system feels bulky or uncomfortable, you may not have an excessive lumbar curve that requires extra support.

Q. How do I adjust the internal inflatable lumbar support system in the CustomAIR®?

A. INFLATE: Rest your lower back against the CustomAIR®. Pump the INFLATION VALVE located on the back bottom, right-hand side. Use your fingers to pump the INFLATION VALVE while gripping the front bottom, right-hand side of the CustomAIR® with the thumb of your right hand.

From full deflation, there are approximately 50 comfort positions for the internal inflatable lumbar support system. When you have reached the most comfortable position for your back, simply stop pumping. The level of air in the lumbar support will remain fixed indefinitely until changed.

DEFLATE: Lean your lower back against the CustomAIR® and fully deflate it by pressing the DEFLATION VALVE located on top of the INFLATION VALVE, on the back bottom, right-hand side of the CustomAIR®.

Q. How is the ObusForme® CustomAIR® Backrest Support cleaned?

A. The ObusForme® CustomAIR® Backrest Support cover is made of synthetic materials. It's easy to clean with a mild soap and a sponge, damp cloth, or soft brush. Do not rub the cover excessively or place it in a washing machine. The cover can be removed and hand washed in cool water. Hang to dry.

To smooth out wrinkled fabric, remove the backrest's cover and iron it using the iron's lowest setting.

Should you wish to purchase a replacement cover, they are available in the same durable, hypoallergenic fabric, at a minimum cost, through your ObusForme® dealer.

Helpful Hint: To increase the air circulation in your backrest, you may remove the thin plastic layer that lies between the foam and the cover. Simply separate the fastener at the back of your backrest, remove the cover, tear off the plastic layer, and replace the cover.

Please Note: The ObusForme® CustomAIR® Backrest Supports may in some cases require time to get used to. Over the years your spine has become less flexible, and needs time to adjust to its natural shape again. If you feel discomfort when using your ObusForme® CustomAIR® Backrest Support for the first time, it is recommended that you use it for shorter periods of time, gradually increasing your usage until you feel comfortable.

If you have a very sore back or small curvature (lumbar area), it is highly recommended that you allow yourself a break-in period of at least 30 days in order to obtain the full benefits of your ObusForme® CustomAIR® Backrest Support.

OBUSFORME® LIMITED WARRANTY

(CANADA Only)

ObusForme® guarantees all items are free from defects in workmanship & materials for a time stated below from the original purchase date. This applies when items are used for the purpose intended. Items will be repaired/replaced with new/refurbished parts/items &/or alternates (our option) if the ORIGINAL purchaser has sent the completed Warranty Registration within 30 days of purchase & includes original receipt & item. Shipping, customs, duties and taxes must be PRE-PAID TO and FROM ObusForme® by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change. This warranty is non-transferable.

WHAT IS NOT COVERED

Wear & tear, aging, foam/item discoloring, odor, flattening, density, variation, leaking, alteration, mishandling, faulty adjustment, misuse, improper care, power damage, accidents, rental use, obsolete items, service by anyone other than ObusForme®, use of any non-ObusForme® authorized parts, shipping damage, over-inflation, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

WARRANTY TIME FRAME

Backrest Frame: Lifetime
(cover, foam, lumbar pad & other Backrest parts/materials are NOT covered)

Seat Frame: 1 year
(cover, foam & other Seat parts/materials are NOT covered)

Back Therapy: 1 year
(Back Therapy includes Backlife, Back Belts, Back Packs, Drivers Seats)

Sleep/Foot/Muscle Therapy: 1 year
(pillow cases are NOT covered)

Electrical Parts: 1 year
(this includes wires, adaptors, plugs and other electrical parts/components)

HOW TO OBTAIN WARRANTY SERVICE
Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please inform Customer Service by:

Mail:
HoMedics Group Canada
344 Consumers Road
Toronto, Ontario, Canada M2J 1P8
Tel: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
www.obusforme.com

Rev. MAY 2010

OBUSFORME®

Corporate Member of ACE
ASSOCIATION OF CANADIAN ERGONOMISTS

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ObusForme® is committed to providing you with optimal relief and comfort. To serve you better in the future, we would like to know if we have fulfilled our commitment. Please complete and return this Questionnaire to help us better meet your needs.

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We aggregate this information and use it internally for research and marketing purposes only. We do not disclose personal information to any third parties.

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