OBUSFORME®

BACKREST SUPPORTS

ObusForme®
Highback Backrest
Support

ObusForme®
Lowback Backrest
Support

ObusForme®
Wideback Backrest
Support

CONGRATULATIONS

on your purchase of an ObusForme Backrest Support. The award-winning ObusForme Backrest Supports provide support and comfort for your back to assist in the prevention and relief of back pain.

The only backrest exclusively endorsed by the Canadian Chiropractic Association, it contains an “S”-shaped frame that conforms to the contours of the back to guide the spine into correct alignment and place you in a proper sitting posture. The ObusForme Lowback Backrest Support, the ObusForme Wideback Backrest Support, and the ObusForme Highback Backrest Support each offer an optional lumbar support Pad.

Lightweight and portable, each backrest support turns any chair into an ergonomic seating system; they can be used in the home, office, vehicle, or anywhere else you sit.

WARNING

Before using your ObusForme Backrest Support in a vehicle, please consult the owner's manual to ensure that you can properly adjust your vehicle's seat and/or headrest after installation of the ObusForme Backrest Support.

QUESTIONS & ANSWERS

Q. How does the ObusForme Backrest Support fasten to a seat or chair?

A. The ObusForme Backrest Support stays in position when the elastic stretch loop is placed or stretched over most chairs or vehicle seats. The stretch loop is not necessary on couches or sofas. To use the ObusForme Highback Backrest Support in a car, truck, or any seat with an opening at the lower back, simply place the bottom ends of the straps through the bottom opening, place the top straps over the top of the seat and secure with buckles.

Q. Why use a lumbar support pad?

A. The lumbar support pad provides added support for the lower spine by filling in the lower inward curve of your back. Lumbar supports can be used for everyday use as extra lower back support or to reduce spinal stress in people with a marked lumbar lordosis or sway-back.

Please Note: It is not necessary to use the lumbar support pad with your ObusForme Backrest Support. If the lumbar support pad feels bulky or uncomfortable, you may not have an excessive lumbar curve that requires extra support.

Q. How is the lumbar support pad positioned?

A. Place your buttocks as far back into the ObusForme Backrest Support as possible. Lean forward and insert the lumbar support pad horizontally across your lower back, above your belt line. Lean back so the hollow in the lower back is firmly supported. Whatever feels comfortable is right for you.

Helpful Hint: To prevent the loss of your lumbar support pad, we recommend you position it inside the backrest's cover. Simply separate the fastener at the back of your backrest and place the lumbar support pad inside the cover towards the front of the backrest. Again, position it where it feels comfortable. Reseal the fastener.

Q. Are there any other uses for the lumbar support pad?

A. Yes, there are a variety of uses for the lumbar support pad:

- Use it as a bolster behind the ObusForme Backrest Support to reduce seat depth of chairs and sofas.
- Try it on its own for additional comfort and support in any chair.
- Try using it as a headrest when using the ObusForme Highback Backrest Support.

Q. How is the ObusForme Backrest Support cleaned?

A. The ObusForme Backrest Support cover is made of synthetic materials. It's easy to clean with a mild soap and a sponge, damp cloth, or soft brush. Do not rub the cover excessively or place it in a washing machine. The cover can be removed and hand washed in cool water. Hang to dry. To smooth out wrinkled fabric, remove the backrest's cover and iron it using the iron's lowest setting. Should you wish to purchase a replacement cover, they are available in the same durable, hypoallergenic fabric, at a minimum cost, through your ObusForme dealer.

Please Note: The ObusForme Backrest Supports may in some cases require time to get used to. Over the years your spine has become less flexible, and needs time to adjust to its natural shape again. If you feel discomfort when using your ObusForme Backrest Support for the first time, it is recommended that you use it for shorter periods of time, gradually increasing your usage until you feel comfortable. If you have a very sore back or small curvature (lumbar area), it is highly recommended that you allow yourself a break-in period of at least 30 days in order to obtain the full benefits of your ObusForme Backrest Support.

Replacement Covers are available for the ObusForme Lowback.

OBUSFORME® LIMITED WARRANTY

(CANADA Only)

ObusForme guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt with the defective product. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change.

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please contact Customer Service by:

HoMedics Group Canada
6460 Kennedy Road, Unit C
Mississauga, Ontario, Canada L5T 2X4
Tel: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
CustomerServiceCAN@obusforme.com
www.obusforme.com

JUL. 2014

Register your product (s) to ensure faster and more efficient customer service. It will allow us to keep you up-to-date with warranty and product information. Please take a few minutes to register online now.

Register at
<http://www.obusforme.com/product-registration/>

OBUSFORME®

Corporate Member of ACE

ASSOCIATION OF CANADIAN ERGONOMISTS

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Canadian
Chiropractic
Association

HB-BLK-CAWB-BLK-CA
LB-BLK-CA/LB-NVY-CA/LB-GRY-CA/LB-BRG-CA
LB-BLK-CB/LB-NVY-CB/LB-GRY-CB/LB-BRG-CB

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