CONTOURED SEAT CUSHION

English, Français

CONGRATULATIONS

Thank you for your purchase of the ObusForme® Contoured Seat Cushion. The unique design of the ObusForme® Contoured Seat Cushion encourages proper alignment of the pelvis and thighs, evenly distributing body weight for extended comfort while sitting. It cushions your tailbone, sitting bones and thighs to help alleviate pressure points.

You can use the ObusForme® Contoured Seat Cushion on its own, or with any ObusForme® Backrest Support to create a complete and comfortable ergonomic sitting experience.

Lightweight and portable, the ObusForme® Contoured Seat Cushion promotes sitting comfort for prolonged periods anywhere – at home, in the office and on the go. The ObusForme® Contoured Seat Cushion is exclusively endorsed by the Canadian Chiropractic Association.

Association of Canadian Ergonomists
Association Canadienne d'Ergonomie

FEATURES


- 1) Contoured sides effectively support your pelvis and hips in a comfortably balanced position.
- 2) Raised center portion separates your legs slightly and rocks your pelvis backward to eliminate excess lumbar curve and reduce spinal strain.
- 3) Cut-out at the center of the seat's posterior reduces pressure on your tailbone.
- 4) Removable front zippered section adjusts the depth to accommodate almost any chair, allowing for proper knee clearance.
- 5) Two tabs at the rear section allow you to securely attach your ObusForme® Contoured Seat Cushion to any ObusForme® Backrest Support.
- 6) Polyurethane foam construction absorbs compression forces and dissipates vibration created in moving vehicles, reducing wear and tear on your spine.

CREATE YOUR OWN ERGONOMIC SEATING SYSTEM

Achieve total comfort when you use your ObusForme® Contoured Seat Cushion with any ObusForme® Backrest Support. These two products complement each other to ensure your total sitting comfort. Each promotes correct posture to help prevent and relieve back pain.

HOW TO ATTACH YOUR OBUSFORME® CONTOURED SEAT CUSHION TO AN OBUSFORME® BACKREST SUPPORT

To attach the ObusForme® Contoured Seat Cushion to an ObusForme® Backrest Support, open the back seam (point A) on the ObusForme® Backrest Support. Insert the two tabs of the Seat (point B). Press the seam to close.

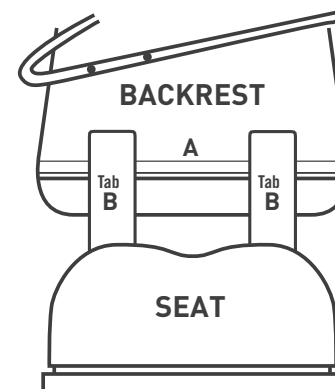Please Note: The ObusForme® Contoured Seat Cushion should be placed in front, and at the base of, the ObusForme® Backrest Support. The ObusForme® Backrest Support should never rest on top of the ObusForme® Contoured Seat Cushion.

USE & CARE TIPS

Adjust the ObusForme® Contoured Seat Cushion To Fit Any Narrow Chair

The ObusForme® Contoured Seat Cushion should never overlap or fold over the front of any chair. If it does, simply remove the front zippered section.

CLEANING YOUR OBUSFORME SEAT

- Sponge wash the cover of your ObusForme® Contoured Seat Cushion;
DO NOT rub excessively.
- DO NOT place the ObusForme® Contoured Seat Cushion or its cover in the washing machine.

Materials

- 100% Polycarbonate frame
- 100% Polyurethane foam fill (CA 117 fire retardant)
- 100% Polyester cover

OBUSFORME® LIMITED WARRANTY

ObusForme guarantees this product will be free from defects in workmanship & materials for a period of one (1) year from the date of original purchase. This guarantee applies when the product is used for the purpose intended. In the advent of a defect as described above, the product will be replaced with a new or refurbished product &/or substitute if the purchaser can provide the original purchase receipt with the defective product. Shipping, customs, duties & taxes must be PRE-PAID TO & FROM ObusForme by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change

WHAT IS NOT COVERED

Wear & tear, aging (including foam discoloration, flattening, density, consistency), accidental damages, alterations, mishandling, faulty adjustment, misuse, improper care, power damage, rental use, discontinued items, service by anyone other than ObusForme, shipping damages, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

HOW TO OBTAIN WARRANTY SERVICE

Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please contact Customer Service.

HoMedics Group Canada
6460 Kennedy Road. Unit C
Mississauga, Ontario, Canada L5T 2X4
Tel: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
CustomerServiceCAN@obusforme.com
www.obusforme.com

Rev. NOV. 2018

Register your product (s) to ensure faster and more efficient customer service. It will allow us to keep you up-to-date with warranty and product information. Please take a few minutes to register online now.

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