
CONTOURED SEAT CUSHION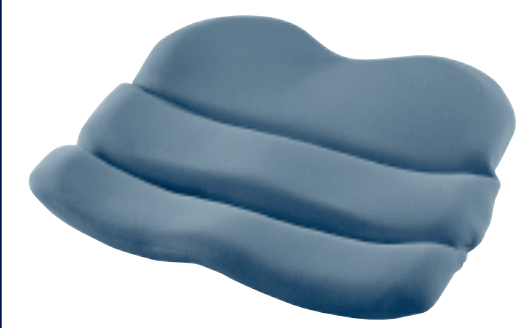

English, Français

CONGRATULATIONS

Thank you for your purchase of the ObusForme® Contoured Seat Cushion. The unique design of the ObusForme® Contoured Seat Cushion encourages proper alignment of the pelvis and thighs, evenly distributing body weight for extended comfort while sitting. It cushions your tailbone, sitting bones and thighs to help alleviate pressure points and assist circulation.

You can use the ObusForme® Contoured Seat Cushion on its own, or with any ObusForme® Backrest Support to create a complete and comfortable ergonomic sitting experience.

Lightweight and portable, the ObusForme® Contoured Seat Cushion promotes sitting comfort for prolonged periods anywhere – at home, in the office and on the go. The ObusForme® Contoured Seat Cushion is exclusively endorsed by the Canadian Chiropractic Association.

FEATURES


- 1) Contoured sides effectively support your pelvis and hips in a comfortably balanced position.
- 2) Raised center portion separates your legs slightly and rocks your pelvis backward to eliminate excess lumbar curve and reduce spinal strain.
- 3) Cut-out at the center of the seat's posterior reduces pressure on your tailbone.
- 4) Removable front zippered section adjusts the depth to accommodate almost any chair, allowing for proper knee clearance and unrestricted circulation.
- 5) Two tabs at the rear section allow you to securely attach your ObusForme® Contoured Seat Cushion to any ObusForme® Backrest Support.
- 6) Polyurethane foam construction absorbs compression forces and dissipates vibration created in moving vehicles, reducing wear and tear on your spine.

CREATE YOUR OWN ERGONOMIC SEATING SYSTEM

Achieve total comfort when you use your ObusForme® Contoured Seat Cushion with any ObusForme® Backrest Support. These two products complement each other to ensure your total sitting comfort. Each promotes correct posture to help prevent and relieve back pain.

HOW TO ATTACH YOUR OBUSFORME® CONTOURED SEAT CUSHION TO AN OBUSFORME® BACKREST SUPPORT

To attach the ObusForme® Contoured Seat Cushion to an ObusForme® Backrest Support, open the back seam (point A) on the ObusForme® Backrest Support. Insert the two tabs of the Seat (point B). Press the seam to close.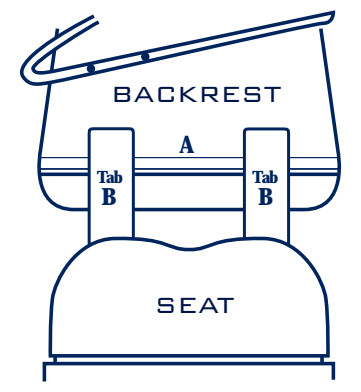


Please Note: The ObusForme® Contoured Seat Cushion should be placed in front, and at the base of, the ObusForme® Backrest Support. The ObusForme® Backrest Support should never rest on top of the ObusForme® Contoured Seat Cushion.

USE & CARE TIPS

ADJUST THE OBUSFORME® CONTOURED SEAT CUSHION TO FIT ANY NARROW CHAIR

The ObusForme® Contoured Seat Cushion should never overlap or fold over the front of any chair. If it does, simply remove the front zippered section.

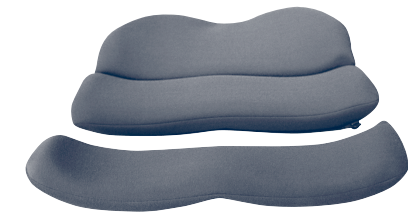CLEANING YOUR OBUSFORME SEAT

- Sponge wash the cover of your ObusForme® Contoured Seat Cushion; DO NOT rub excessively.
- DO NOT place the ObusForme® Contoured Seat Cushion or its cover in the washing machine.

MATERIALS

- 100% Polycarbonate frame
- 100% Polyurethane foam fill (CA 117 fire retardant)
- 100% Polyester cover

OBUSFORME® LIMITED WARRANTY

(CANADA Only)
ObusForme® guarantees all items are free from defects in workmanship & materials for a time stated below from the original purchase date. This applies when items are used for the purpose intended. Items will be repaired/replaced with new/refurbished parts/items &/or alternates (our option) if the ORIGINAL purchaser has sent the completed Warranty Registration within 30 days of purchase & includes original receipt & item. Shipping, customs, duties and taxes must be PRE-PAID TO and FROM ObusForme® by the PURCHASER. This warranty gives you rights that vary by province/state. This warranty may change. This warranty is non-transferable.

WHAT IS NOT COVERED

Wear & tear, aging, foam/item discoloring, odor, flattening, density, variation, leaking, alteration, mishandling, faulty adjustment, misuse, improper care, power damage, accidents, rental use, obsolete items, service by anyone other than ObusForme®, use of any non-ObusForme® authorized parts, shipping damage, over-inflation, neglect, items sold 'as is' or damage due to natural acts are NOT covered.

WARRANTY TIME FRAME

- Backrest Frame: Lifetime**
(cover, foam, lumbar pad & other Backrest parts/materials are NOT covered)
- Seat Frame: 1 year**
(cover, foam & other Seat parts/materials are NOT covered)
- Back Therapy: 1 year**
(Back Therapy includes Backlife, Back Belts, Back Packs, Drivers Seats)
- Sleep/Foot/Muscle Therapy: 1 year**
(pillow cases are NOT covered)
- Electrical Parts: 1 year**
(this includes wires, adaptors, plugs and other electrical parts/components)

HOW TO OBTAIN WARRANTY SERVICE
Please obtain a Return Authorization Number & instructions prior to sending your item or it may be denied. Please inform Customer Service by:

Mail:
HoMedics Group Canada
344 Consumers Road
Toronto, Ontario, Canada M2J 1P8
Tel: (416) 785-1386 Fax: (416) 785-5862
Toll Free: 1-888-225-7378
8:30 a.m. to 5:00 p.m., Mon - Fri EST
www.obusforme.com

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ObusForme® is committed to providing you with optimal relief and comfort. To serve you better in the future, we would like to know if we have fulfilled our commitment. Please complete and return this Questionnaire to help us better meet your needs.

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Male / Homme _____ Female / Femme _____ Age / Age: _____
Occupation / Profession: _____

1. Which ObusForme® product did you purchase? / Quel produit ObusForme avez-vous acheté?
Product Description: / Description du modèle: _____
(and model number if applicable)
(Example: ObusForme® Lowback Backrest Support, Burgundy) / (Exemple: Le Dossier ObusForme®, Bourgogne)

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